OPERATING INSTRUCTIONS

EN

1. Twenty to thirty minutes before use, remove the product box from the refrigerator, remove the blister pack from the box and allow the syringe to come to room temperature. Do not use if the sterile package appears to be damaged.

2. Peel off the blister backing. (Fig. 1)

3. While holding the blister open side down, bend the blister and allow the syringe to fall gently onto the sterile surface. (Fig. 2)

Alternatively, hold the blister open side up and bend back the blister until the barrel's luer end is exposed. Gripping the luer end of the barrel, remove the syringe from the blister. **Do not remove the syringe from the plunger end.**

4. Remove the tip cap (Fig. 3.1, Fig. 3.2) from the syringe and attach the cannula (Fig. 4.1, Fig. 4.2). **Attention:** Do not apply pressure to the plunger rod while the cannula is being affixed. Verify that the cannula is properly locked to the Luer Lock Adaptor (LLA). Do not overtighten the LLA; this can lead to loosening of the LLA from the barrel.

5. Apply gentle pressure to the plunger in order to expel air from the cannula and verify that the syringe is operating properly. (Fig. 5)

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6309501203

BIOLON

LOT	Batch code
	Use by (YYYY-MM-DD : year-month-day)
REF	Catalog number
MD	Medical device
	Keep dry
STERILE EO	Sterilized by ethylene oxide
STERILE A	Sterilized by aseptic techniques
	Temperature limit
STERILE	Do not resterilize
DBS	Double sterile barrier system

	Caution, see instructions for use
	See instructions for use
	Do not reuse
	Protect from light
	Do not use if the packaging has been opened or damaged
	Keep dry
	Temperature limit
	Sterilized by ethylene oxide
	Sterilized by aseptic techniques
	Manufacturer

BioLon®

MANUFACTURED BY

Nordic Group B.V.
Siriusdreef 41,
2132 WT Hofddorp,
The Netherlands

6309501203 Issue date: xx/20xx

CE 2797